

RoHS

1620

SPECIFICATIONS

- Disposable Blood Pressure Sensor
- AAMI Specifications
- Low Cost Disposable
- Dielectric Gel Barrier
- Fully Tested & Calibrated

The 1620 is a fully piezo resistive silicon pressure sensor for use in invasive blood pressure monitoring. The sensor is designed to be used with automated assembly equipment and can be dropped directly into a customer's disposable blood pressure housing. The sensor is designed to meet the requirements as described in the Association for the Advancement of Medical Instrumentation (AAMI) specification for Blood Pressure Transducers.

The pressure sensor consists of a pressure sensing element mounted on a ceramic substrate. Thick film resistors on the ceramic substrate are laser trimmed for compensation and calibration. A plastic cap is attached to the ceramic substrate to provide an easy method of attachment to the customer's assembly and protection for the sensing element. A dielectric gel is placed over the sensor to provide electrical and fluid isolation.

The products are shipped on a tape and reel. Performance characteristics and packaging can be easily tailored on a special order basis to meet the requirements of specific customers.

iSweek www.isweek.com

Add: 16/F, Bldg. #3, Zhongke Mansion, No.1 Hi-Tech S. Rd, Hi-Tech Park South, Shenzhen, Guangdong, 518067 P.R.ChinaTel: + 86-755-83289036Fax: + 86-755-83289052E-mail: sales@isweek.com

FEATURES

- Low Cost, Medical Applications
- Small Size and Reliable Performance
- Gel Isolation for Liquids
- Operates from 10°C to 40°C
- Compatible for Automated Assembly
- 1% Accuracy for Replacements
- 5.0 uV/V/mmHg Sensitivity
- Customization for OEM Applications

APPLICATIONS

- Surgical Procedures
- Intensive Care Units
- Infusion Pumps
- Kidney Dialysis Machines
- Vacuum Assisted Birth
- Intrauterine Monitoring

STANDARD RANGES

Range	mmHg
-50 to 300	•

PERFORMANCE SPECIFICATIONS

Supply Voltage: 6.0 Vdc

Ambient Temperature: 23°C (unless otherwise specified)

PARAMETERS	MIN	TYP	МАХ	UNITS	NOTES
Operating Pressure Range	-50		300	mmHg	
Over Pressure	125			psi	
Zero Pressure Offset	-20		20	mmHg	
Sensitivity	4.95	5.00	5.05	uV/V/mmHg	
Calibration	97.5	100	102.5	mmHg	1
inearity and Hysteresis (-30 to 100 mmHg)	-1		1	mmHg	2
inearity and Hysteresis (100 to 200 mmHg)	-1		1	% Output	2
inearity and Hysteresis (200 to 300 mmHg)	-1.5		1.5	% Output	2
nput Impedance					
1620-A	1200		3200	Ω	
1620-E	1200		3200	Ω	
1620-F	325		375	Ω	
Dutput Impedance	285		315	Ω	
Dutput Symmetry	0.95		1.05	Ratio	3
Supply Voltage	2	6	10	Vdc or Vac rms	
Risk Current (@ 120 Vac rms, 60Hz)			2	uA	
Narm-Up Time		5		Seconds	
Frequency Response		1200		Hz	
Offset Drift			2	mmHg	4
Thermal Span Shift	-0.1		0.1	%/°C	5
Thermal Offset Shift	-0.3		0.3	mmHg/ → C	5
Phase Shift (@ 5KHz)			5	Degrees	
Light Sensitivity (3000 Foot Candle)		1		mmHg	
Defibrillator withstand (400 joules)	5			Discharges	6
Sterilization (ETO)	3			Cycles	7
Dperating Temperature	10		40	→C	•
Storage Temperature	-25		+70	→C	
Humidity (External)		densing)	+70		
Derating Product Life	10-90% (non-condensing) 168 Hours				
Shelf Life	5 Years				
	10000Vdc				
Dielectric Breakdown					
Aedia Interface	Dielectric Gel				
Volume Displacement Neight	4.5 x 10 ⁻⁴ mm ³ 0.5 grams				

Notes

1. Output of sensor with no pressure applied and a 150K Ohms resistor placed across +SUPPLY to +OUTPUT.

2. Best fit straight line.

3. Defined as common mode symmetry between any output and supply terminal.


- 4. Over an 8 hour time period after a 10 minute warm-up.
- 5. Over operating temperature range 10-40°C with respect to 23°C.
- 6. One discharge per minute performed by customer.
- 7. Sterilization performed by customer.
- 8. This model is not suitable for surface mount operation.

Sweek www.isweek.com

Add: 16/F, Bldg. #3, Zhongke Mansion, No.1 Hi-Tech S. Rd, Hi-Tech Park South, Shenzhen, Guangdong, 518067 P.R.China


Tel: + 86-755-83289036 Fax: + 86-755-83289052

Sweek.com

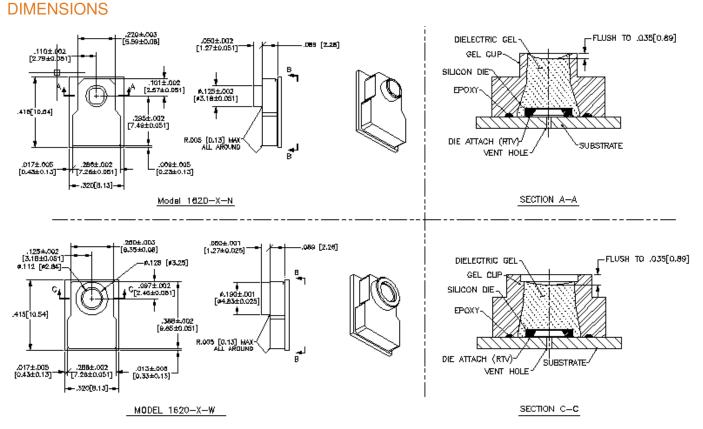


^{*}Dimensions indicated to not include flares.

SHIPPING INFORMATION

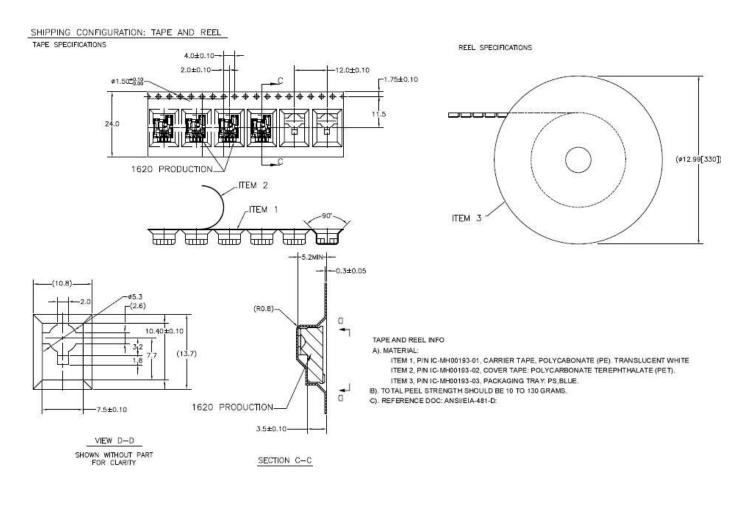


İSweek www.isweek.com Add: 16/F, Bldg. #3, Zhongke Mansion, No.1 Hi-Tech S. Rd, Hi-Tech Park South, Shenzhen, Guangdong, 518067 P.R.China Tel: + 86-755-83289036 Fax: + 86-755-83289052 E-mail: sales@isweek.com

STANDARD PAD CONFIGURATIONS

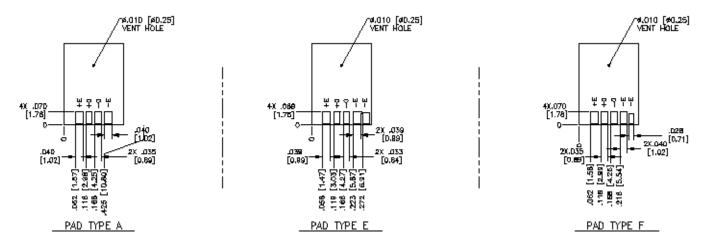

CONNECTIONS

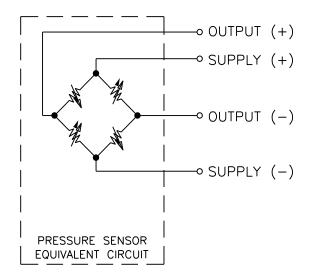

İSweekwww.isweek.comAdd: 16/F, Bldg.#3, Zhongke Mansion, No.1 Hi-Tech S. Rd, Hi-Tech Park South, Shenzhen, Guangdong, 518067 P.R.ChinaTel: + 86-755-83289036Fax: + 86-755-83289052E-mail: sales@isweek.com

ORDERING INFORMATION

1620	-	A	-	N	Т
Model	-	Pad Type	-	Pressure Port Type	Shipping
1620	-	 A = Pad Type "A" E = Pad Type "E" F = Pad Type "F" 	-	N = Narrow Hole W = Wide Hole	T = Tape and Reel

İSweekwww.isweek.comAdd: 16/F, Bldg.#3, Zhongke Mansion, No.1 Hi-Tech S. Rd, Hi-Tech Park South, Shenzhen, Guangdong, 518067 P.R.ChinaTel: + 86-755-83289036Fax: + 86-755-83289052E-mail: sales@isweek.com